


DIRECTIONS FOR USE


READ ALL INSTRUCTIONS CAREFULLY BEFORE USING

ATTENTION: This poster provides a quick reference for proper use of Qivi™. The Qivi - Male External Urine Management device is indicated for non-invasive urine management in semi and non-ambulatory adolescent or adult male patients by diverting and collecting urine away from the patient's body. Please consult product IFU for further information.

PREPARATION

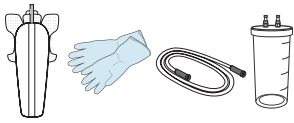

Gather the necessary supplies and set up station close to the patient

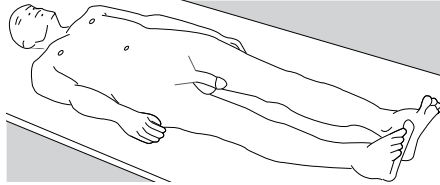

Position the patient in supine position with their legs slightly spread apart

CONTRAINDICATIONS

DO NOT use the product on patients who:

- Have urinary retention
- Have female anatomy
- Have allergy to adhesive or any other device material
- Have glans penis and/or penile shaft complications, including but not limited to the following:
 - Open lesions
 - Skin irritations
 - Ulcers
 - Inflammation


PRECAUTIONS AND OBSERVATIONS

1. Not recommended for patients who are:
 - Agitated, restless or uncooperative and may remove the product while in use
 - Experiencing skin irritation or breakdown at the site
2. Not to be used for stool collection
3. Proceed with caution in patients who have undergone recent surgery of the external urogenital area
4. Always assess skin integrity and perform perineal care prior to placement of this product
5. Maintain suction until the product is fully removed from the patient to avoid any urine backflow


WARNINGS

1. Do not use the Qivi - Male External Urine Management device with any other product that compresses the device or does not allow for sufficient fluid/air flow
2. Prepare skin prior to use, be gentle during application and removal to avoid any potential skin injury
3. Discontinue use if an allergic reaction is observed
4. After use, this product may be a potential biohazard, dispose it in accordance to your institutional protocol

PLACEMENT

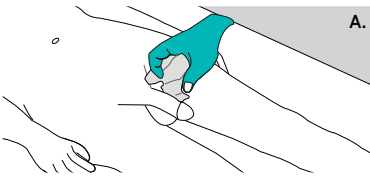


Clean and dry suprapubic region, perform perineal care and assess skin integrity. It is advisable to trim pubic hair to improve product performance

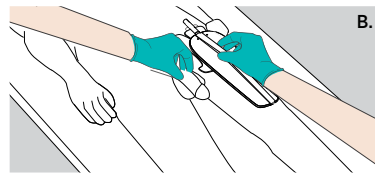

Hold the penile shaft and push down against abdominal wall to expose full length. Slide the penile shaft until the orifice is flush at the base of the shaft




Hold device in place, apply adhesive from top to bottom smoothly over the suprapubic region

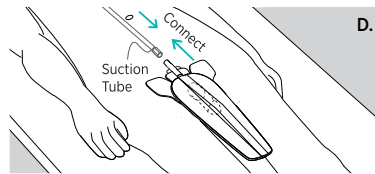


Connect suction tube to the device and set vacuum at 40 mmHg of continuous pressure.

MANAGEMENT AND REMOVAL

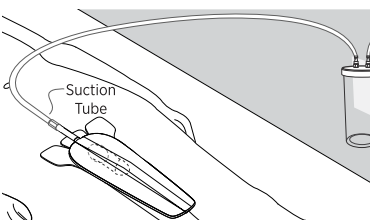

Ensure tubing is connected at all times while device is in use. Replace the device every 12 to 24 hours or every shift change

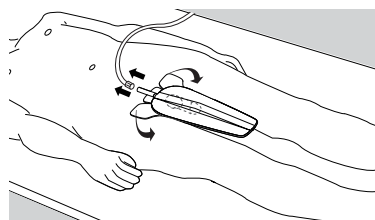

Gently peel the adhesive in the direction of suprapubic hair growth. Disconnect the suction tube and dispose of the device as per institutional protocol



Product use video

iMEDicare
Pelvic Health Naturally...



Connect With Us!

contact@imedicare.co.uk
+44 (0) 1923 23 77 95
www.MyPelvicHealth.co.uk
iMEDicare LTD
Unit 11, Shakespeare Street
Watford Herts WD24 5RR

conSURE
MEDICAL

Lewisville, TX 75067
© CM Technologies, Inc. All rights reserved