



PREPARATION OF DEVICE

1. In addition to the QiVi® - Female External Urine Management device, suction canister/bag and associated tubing will be required.
2. Wear gloves before handling the patient.
3. Unpack the device and set up station near the patient.
4. Connect the canister to wall suction and set pressure at a minimum 40mmHg of continuous suction. Connect the tube with the canister.

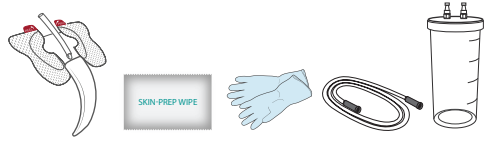


PREPARATION OF PATIENT

1. Position the patient in supine position with legs slightly spread apart.
2. Position yourself such that the patient's urogenital region is clearly visible and directly accessible.
3. Perform perineal care and assess skin integrity (document per institutional protocol).
4. It is advisable to trim pubic hair to improve product performance.

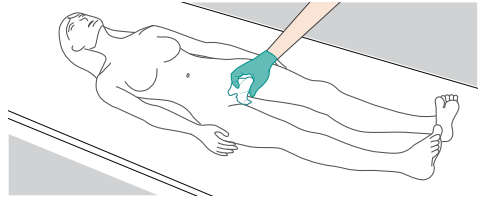


PLACEMENT OF DEVICE

1. Hold the device upright with the cranial tube on top.
2. Align the distal end at the perineum and place the device taut over the labia with the collection chamber contouring the urethral opening.
3. While holding the device in place, use the peel tabs to remove the adhesive liner and place the adhesive patch on the suprapubic region.
4. Connect the suction tube with the QiVi® device. Ensure all connections are secure.
5. Return the patient to natural position. Ensure the suction tube does not run under the patient.

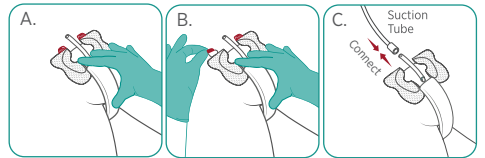

MAINTENANCE OF DEVICE

1. Perform skin assessment periodically as per institutional protocol. Ensure tube is connected at all times while device is in use.
2. Assess device periodically to ensure proper placement, particularly after turning or repositioning the patient. Replace device every 12 to 24 hours or if contaminated due to fecal exudate or other fluids that may hamper device performance. Dispose the device as per institutional protocol.
3. Replace tube and empty and/or replace suction canister as per institutional protocol.

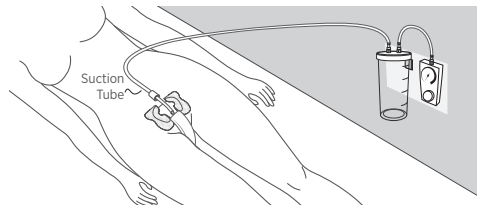

REMOVAL OF DEVICE

1. Position the patient in supine position with legs slightly spread apart.
2. Position yourself such that the patient's urogenital region is clearly visible and directly accessible.
3. Starting from the proximal end, gently peel the adhesive patch and remove the device.
4. Disconnect the device from the suction tube and discard device as per institutional protocol.
5. Assess skin integrity and perform perineal care if required.

