

FOR BEST RESULTS:

Clean application site with an alcohol wipe and/or non-moisturizing soap prior to application.

Contents:

- 1 Alcohol Wipe*
- 1 BioPlus+™ Barrier Wipe
- 1 Hydrocolloid Universal Securement Device

For use on intact skin.

If redness or irritation occurs, stop use and consult a healthcare practitioner.

For more info, please contact your local iMEDicare representative, or visit www.MyPelvicHealth.co.uk, info@mypelvichealth.co.uk or call 01923237795 (8am to 5pm Mon-Fri).

See reverse side for CathGrip complete application and removal instructions.



BioDerm XLS (Oval)
NHS Code# 20026-10
Pack of 10
CathGrip® supplied separately

UKRP: Advena Limited
Pure Offices, Plato Close, Tachbrook Park
Warwick, CV34 6WE, United Kingdom
Registered in England & Wales No. 3517275
T: +44 1926 800153
E: info@advenamedical.com

Contraindications:

Absolute (When the product should not be used at all)


- Demonstrated sensitivity to hydrocolloid (rare)
- Anatomic abnormality (severe acquired hypospadias, urethral fistula, penile shaft erosions)

Relative (When the product can be used after healing)

- Unhealed wound at application site (wait to heal)
- Active rash at application site (wait to heal)
- Active inflammation or infection of the glans, prepuce or urethra (wait to heal)
- Purulent or bloody urethral discharge (wait to heal)

Any serious incident while using the device should be reported to the manufacturer and competent authority.

Using Men's Liberty with a diaper or other absorbent products may increase your risk of skin irritation or infection. If redness or irritation occurs, discontinue use and consult your healthcare professional.

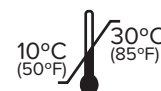

Bioderm, Inc.
Largo, FL 33773, USA
Support: (800) 864-6755
CustomerCare@BioDermInc.com
www.BioDermInc.com



U.K. Distribution:
iMEDicare Ltd
Unit 11 Shakespeare Indus. Est.
Watford HERTS WD24 5RR
www.MyPelvicHealth.co.uk
info@mypelvichealth.co.uk
T: 01923237795



Authorized Representative:
Advena Ltd.
Tower Business Centre, 2nd Flr,
Tower Street, Swatara, BKR 4013
Malta

NOT MADE WITH NATURAL RUBBER LATEX

PART# 20026-10

LT 95205 F

iMEDicare Ltd, Item Number BIO2102610

©2020 BioDerm, Inc. All rights reserved. BioDerm® and CathGrip® are registered trademarks of BioDerm, Inc. Men's Liberty Acute™, BioPlus+™, FreeDerm™ and KindKlamp™ are trademarks of BioDerm, Inc. US. Patent #8,551,062 and #10,022,261. Global Patent EP (UK and NL) 2501429. International patents pending.

Application Tips



Make sure the patients' anatomy is **COMPLETELY DRY** before applying Men's Liberty™ Acute.



Do **NOT** use any cleansing wipes, moisturizing creams, lotions or oils on the penis prior to application.



DO NOT use other products between Men's Liberty™ Acute applications (i.e. cleansing or chlorhexidine wipes). Excess skin moisture caused by other incontinence products will interfere with Men's Liberty Acute's seal, shortening wear time.



Do **Not** connect to suction.



Wear time. The first application for each patient will last shorter than the subsequent applications of the product. You can expect the first application to last 6-12 hours and subsequent applications to last 24-48 hrs. This is due to the hydrocolloid material that absorbs underlying moisture in the skin.

Product is not reusable and could cause product to malfunction or risk of infection if reused. Once removed, dispose of in a trash can.

Removing Men's Liberty Acute

FreeDerm Instantly removes Men's Liberty Acute with a gentle, quick drying, no washing, no-sting, skin friendly formula. Remove and replace Men's Liberty Acute when the adhesive turns a white color, when it is loose, or when it has been soiled. We recommend using FreeDerm adhesive remover to remove Men's Liberty Acute. You can also use warm water for removal by soaking the petals or wrapping a washcloth around the device until the hydrocolloid turns milky white and rolls gently from the skin.

To Remove with FreeDerm:

Wipe or spray at the edge of the adhesive, lifting away from the skin as it releases. Continue until entire adhesive has been removed. Depending on the size of the adhesive, this may require more than one spray or ampoule.



Sold separately in a box of 30 ampoules (SKU: 52203). Also available as 1 oz. pump spray (SKU: 52200).

Application Instructions



1 Wash & Dry

Always begin with a clean and dry penis. Keep foreskin back completely throughout application. To begin, wipe the head of the penis with the alcohol wipe and allow to dry.

2 Pre-Application Prep

Wipe the head of the penis with BioPlus+™ wipe. Allow 15-20 seconds to dry until the skin is no longer tacky.

3 Positioning & Application

Position Men's Liberty Acute with the vent facing up. Remove release papers around the petals and attach the faceplate to the tip of the penis with the center opening directly in line with the urinary opening. Smooth down petals.



4 Secure and Seal

Apply the faceplate seal around the head of the penis, flat and wrinkle free. Be sure the adhesive material is pressed into all of the ridges and grooves in the skin. Press and hold for 10-15 seconds to secure the seal. After catheter application on uncircumcised patients, allow the foreskin to fall back to its natural position.


5 Connect to Leg or Bedbag

Connect to Leg or Bedside Bag and be sure to keep below the bladder or bed to ensure drainage. Do not twist or kink the tube. **For best results, use CathGrip® Double Strap to secure tube and reduce tension.**

To place an order, please contact your local iMEDicare representative, or visit www.MyPelvicHealth.co.uk, orders@imedicare.co.uk or call 01923237795 (8am to 5pm Mon-Fri).

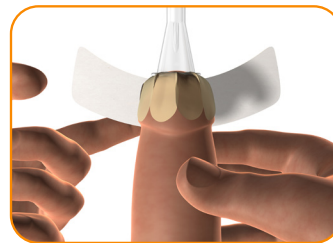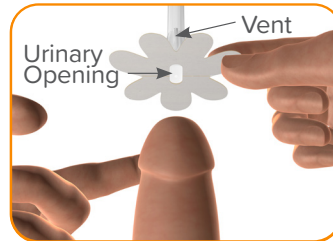


Fit the Bed Bag or Leg Bag of your choice

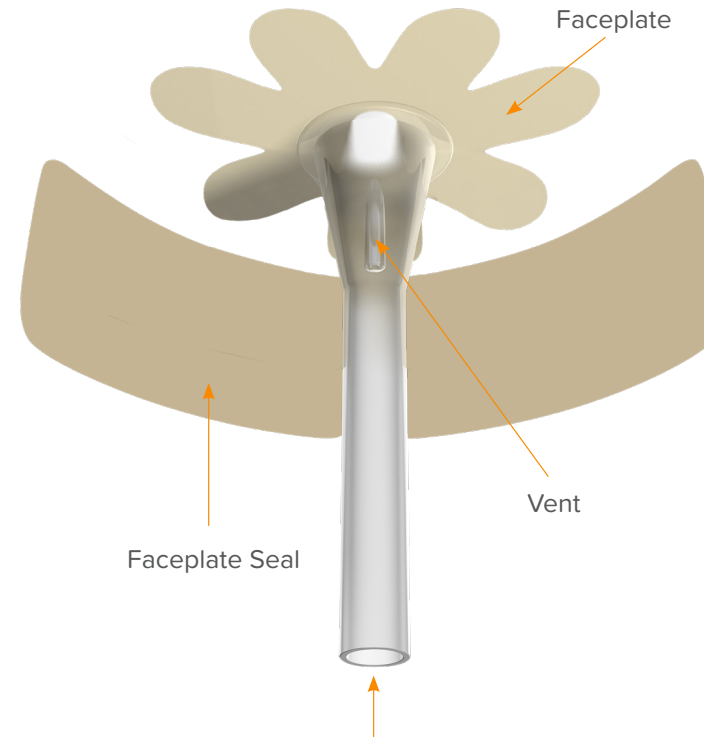

One size fits most incontinence patients

CathGrip® Double Strap

CathGrip® Double Strap is the recommended device to anchor Men's Liberty Acute

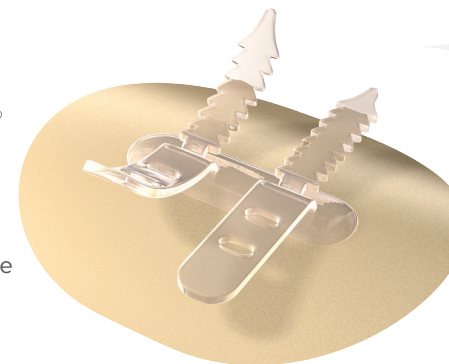


Applying CathGrip®




STEP 1:

Clean entire application site with alcohol wipe and allow to dry.

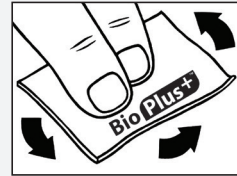


STEP 2:

Wipe entire application site with BioPlus+™. Allow 15+ seconds to dry.

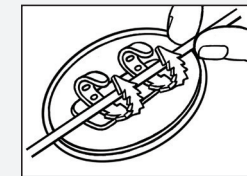

STEP 3:

Remove the release papers and press to the skin to seal.

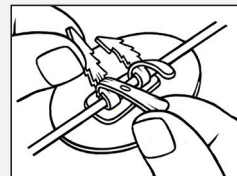

STEP 4:

Rest the tube on the soft, no-slip grips. Pull tight to secure.

Removal

Frederm™ is recommended to remove CathGrip. Begin by soaking with warm water or wet washcloth until CathGrip is milky white and rolls gently from the skin. Spray Frederm at edge of the adhesive, lifting gently from the skin as you go. Continue to spray under the adhesive until completely removed. Never pull or tear from the skin.

ATTENTION: Following application, leave a one inch space around the CathGrip® device when using any cleansing products, creams or lotions. Contact with the hydrocolloid will result in early dislodgement of the securement device.