


uricap Female

Product
manufactured
by TillaCare

tillaTM
care

What is UriCap Female?

UriCap Female is an Innovative and Safe solution for urine incontinence management in women, the preferred alternative to invasive catheters and wet single use adult disposable pads.

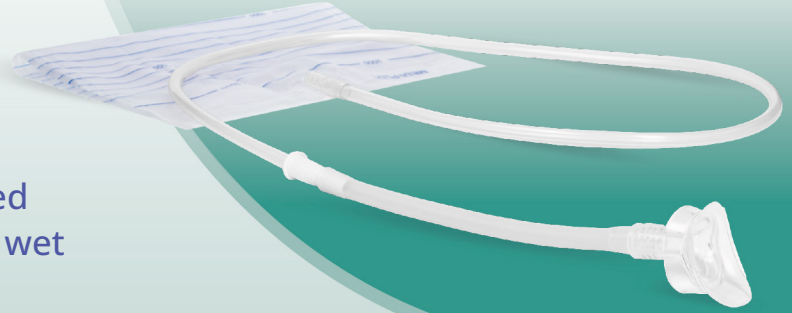

Key benefits of the UriCap Female:



Is made of soft medical grade silicone, connects to a standard urine bag, and relies on natural adhesion and gravity.



Does not require any use of external suction or glue.



Is disposable (single-use), affordable, and comfortable with up to 24-hour use.



Forms a natural seal surrounding the urethral opening preventing leakage and protecting against fecal contamination.


How does it work?

The UriCap Female stays in place naturally by relying on three factors:

- 1 A natural adhesion is formed by the continuous surface-to-surface contact between the surface under the UriCap device and the skin surrounding the urethra. The adhesion allows for sideways motion without friction, all minimizing leakage, discomfort and risks for wounds.
- 2 The inner and outer labial folds enclosing the area contribute to keeping the surfaces close together and in place.
- 3 Small or larger voids aid in enhancing the natural adhesion forces of the seal as they bring the surfaces closer together. This occurs gradually following application of the UriCap.

Why use the UriCap Female?

- ✓ Safe alternative to catheterization
- ✓ Ideal for obtaining a clean catch urine specimen
- ✓ Supports non-invasive monitoring of urine output
- ✓ No suction needed, no sterile procedure needed
- ✓ Prevention of dermatitis
- ✓ No sterile procedure required, quick and easy to apply and remove
- ✓ Reduces demand of heavy lifting
- ✓ 15x ROI after reduction of 300 catheters
- ✓ Decreases waste and need for storage and shelf space

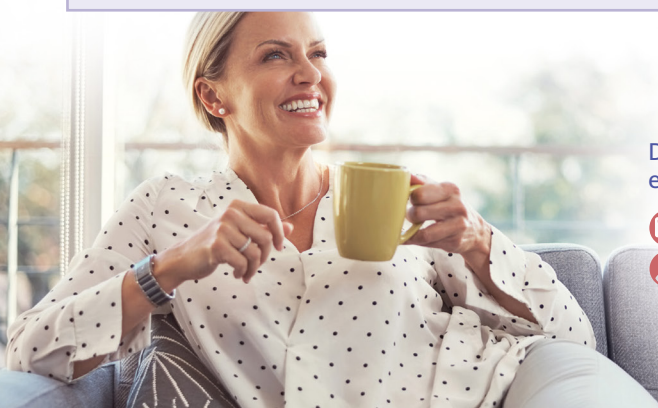

Distributed in the UK and Ireland
exclusively by iMEDicare LTD

 contact@imedicare.co.uk
 +44 (0) 1923 237795

iMEDicare
Pelvic Health Naturally...




Is the UriCap Female appropriate for your patient?


This product is appropriate for your patient when:

- ✓ She is experiencing UI and mobility limitations (bed or chair bound)
- ✓ Needing to obtain a clean-catch urine sample
- ✓ At risk of falling from frequent urination/toilet use at night
- ✓ Seeking drier nights and higher quality sleep


As a clinician UriCap Female offers the possibility of avoiding catheters for:



Patient urine output monitoring.



Urine specimen collection in female patients experiencing incontinence.



Safe and non-suction based urine collection.

When is the UriCap Female not suitable?

UriCap Female should not be used on women:

- ✓ Sensitive or allergic to silicone
- ✓ With vaginal discharge, itch, inflammation, or other skin conditions
- ✓ Women with urinary retention
- ✓ With pelvic prolapse (stage 2 and up)
- ✓ 18 years or younger

Studies

Year & Study	Population	Results and conclusions
2017 UriCap feasibility Clinical Study	Multisite, 3 nursing homes, 30 female participants with UriCap used for 756 nights.	<ul style="list-style-type: none"> • All urine in bag due to surface adhesion, no vacuu in play. • Stayed on and sealed the periurethral area from contamination. No adverse events. • Design modified for serial production and ease of use.
2019 Safety, Performance and Usability	Multisite, 2 Hospitals, 14 participants with 149 UriCap applications, continuous use up to 30 days.	<ul style="list-style-type: none"> • Up to 34 days usage. • Safe to use with daily change. • Easy to apply with a few mild side effects. • Collected expected amount of urine in 72%. • Design modified to further increase softness of device. • Improved usability with prolongation of the extension tube. • Adds securements device to leg to further improve stability of device
2020 TillaCatch Urine Collection for Culture	One Hospital Ambulatory Urology Department, 30 participants, 29 UriCap urine collections for culture included.	<ul style="list-style-type: none"> • 28 our of 29 samples collected without contamination. • The device was safe, easy to use with high user satisfaction. No adverse events. • The Device and method served as an excellent non-invasive, low-risk alternative for urine sampling for culture in hospital and ambulatory settings.
2023–2028 Better Care for Women with Urinary Incontinence	Multisite, control arm, in hospitals and long term care to measure reduction of catheterizations and UTIs.	<ul style="list-style-type: none"> • Study in progress through 2028.

Ordering system

MPC	Brand Name	Description	GTIN Number*	Primary Unit of Measure
KIT000011-35	UriCap Female Starter Kit	Each kit is for one daily use and contains: <ol style="list-style-type: none"> 1. UriCap Female device 2. Extension Tube SX-L 3. Sterile Lubricating Jelly 4. Alcohol Prep Pad 5. Securement Device 	0729010 9270081	Box of 35
KIT00012	UriCap Female Change Kit	Each kit is for one daily use and contains: <ol style="list-style-type: none"> 1. UriCap Female device 2. Sterile Lubricating Jelly 	0729010 9270098	Box of 70
KIT00013-30	UriCap Female Base Kit	Each kit is for one daily use and contains: <ol style="list-style-type: none"> 1. UriCap Female device 2. Extension Tube SX-L 3. Sterile Lubricating Jelly 	0729010 9270173	Box of 30
HRD00008	UriCap Urine Bag 7ID	Urine Bag for use with UriCap – 2000ml	0729010 9270128	Box of 35
HRD00006	UriCap Securement Device	Medical Catheter stabilization device manufactured by Greystone	0729010 9270135	Box of 50

* GTIN (Global Trade Item Number – GS1 Barcode) MANDATORY FOR MEDICAL DEVICES WHERE UDI CARRIER IS GS1.